

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

FILAPROP SURGICAL SUTURE

**Nome Técnico:** 2501399 - Fio de Sutura  
**Produto:** FILAPROP SURGICAL SUTURE  
**Registro ANVISA nº:** 10360810012

### FIO DE SUTURA NÃO ABSORVÍVEL ESTERILIZADO



#### Detentor Registro/Importador:



DOC MED Comércio Importação E Exportação Ltda.  
CNPJ: 66.877.184/0001-80  
End: Alameda dos Maracatins nº 992, 8º andar, São Paulo, SP  
Tel: (11) 2506-3624  
[www.merillife.com](http://www.merillife.com)  
Responsável Técnico: Caio de Macedo Matos

#### Fabricante:



Meril Endo Surgery Pvt. Ltda  
Survey nº 135/2/B e 174/2, 3º andar, E1-E3,  
Meril Park \_ Bilakhia Muktanand Marg, Chala, Vapi, Dist: Valsad – 396191 Gujarat, India.  
[www.merillife.com](http://www.merillife.com)

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

#### I - DESCRIÇÃO DETALHADA DO PRODUTO MÉDICO

FILAPROP™ sutura é uma sutura cirúrgica estéril monofilamento, sintética e não absorvível, composta por um estereoisômero de polipropileno isolático cristalino, uma poliolefina linear sintética. A fórmula molecular do polipropileno é (C<sub>3</sub>H<sub>6</sub>)<sub>n</sub>. As suturas FILAPROP™ são tingidas com azul ftalocianina (Número do Índice de Cor: 74160).

FILAPROP™ está disponível numa variedade de tamanhos e comprimentos, ligada a agulhas padrão, de aço inoxidável, de vários tipos e tamanhos.

FILAPROP™ está em conformidade com os requisitos da Farmacopeia dos Estados Unidos para “Fio de Sutura Cirúrgica Não Absorvível” e a Farmacopeia Europeia para “Fio Monofilamento Sintético Estéril Não Absorvível”. No entanto, pode ser ligeiramente sobredimensionada em diâmetro para a USP para alguns tamanhos de sutura.

O Filaprop Surgical Suture é fornecido estéril, esterilizado por Óxido de Etileno.

**Esse é um produto médico enquadrado como proibido reprocessar.**

#### MODELOS QUE COMPÕEM A FAMÍLIA DE FIOS DE SUTURA

Modelo do Produto	Descrição	Diâmetro da sutura (USP)	Comprimento sutura (cm)	Comprimento Agulha (mm)	Formato da Agulha	Tipo de Agulha
PPG40846	POLYPROPYLENE BU 4-0 X90-17MM HC RB DNTP	4-0	90	17	Half Circle	Round Body, Double Needle, Teflon Pledget
PPL012989	POLYPROPYLENE BU 1 X90-90MM CU CT	1	90	90	Half Circle	Cutting Needle
PPL01834	POLYPROPYLENE BU 1 X150-48MM HC RB LOOP	1	150	48	3/8 Circle	Round Body, Loop
PPL01840	POLYPROPYLENE BU 1 X 70-30MM HC RB	1	70	30	Half Circle	Round Body
PPL01843	POLYPROPYLENE BU 1 X 70-40MM HC RB	1	70	40	Half Circle	Round Body
PPL018435	POLYPROPYLENE BU 1 X 100-40MM HC RB	1	100	40	Half Circle	Round Body
PPL018450	POLYPROPYLENE BU 1 X 90-45MM HC TC	1	90	45	Half Circle	Round Body
PPL018475	POLYPROPYLENE BU 1 X 90-50MM HC RC	1	90	50	Half Circle	Taper Cut

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL01883	POLYPROPYLENE BU 1 X 100-45MM HC RC	1	100	45	Half Circle	Reverse Cutting
PPL1001713	POLYPROPYLENE BU 10-0 X 90-16MM ST CT	10-0	90	16	Half Circle	Reverse Cutting
PPL1001777	POLYPROPYLENE BU 10-0 X 90-6.3MM CU SM	10-0	90	6,3	Straight	Cutting Needle
PPL1002795	POLYPROPYLENE BU 10-0 X 13-06MM CU RB	10-0	13	6	3/8 Circle	Spatulaed Musles Needle
PPL100830	POLYPROPYLENE BU 0 X 75-30MM HC RB	0	75	30	3/8 Circle	Round Body
PPL102815	POLYPROPYLENE BU 0 X 90-30MM CU RC	0	90	30	Half Circle	Round Body
PPL102820	POLYPROPYLENE BU 0 X 90-26MM HC RC	0	90	26	3/8 Circle	Reverse Cutting
PPL102825	POLYPROPYLENE BU 0 X 90-36MM HC RC	0	90	36	Half Circle	Reverse Cutting
PPL10830	POLYPROPYLENE BU 0 X 70-30MM HC RB	0	70	30	Half Circle	Reverse Cutting
PPL108424	POLYPROPYLENE BU 0 X 75-37MM HC RB	0	75	37	Half Circle	Round Body
PPL108434	POLYPROPYLENE BU 0 X 75-40MM HC RB	0	75	40	Half Circle	Round Body
PPL108454	POLYPROPYLENE BU 0 X 75-48MM HC CT	0	75	48	Half Circle	Round Body
PPL10846	POLYPROPYLENE BU 0 X 90-30MM HC RB	0	90	30	Half Circle	Cutting Needle
PPL108470	POLYPROPYLENE BU 0 X 90-40MM HC RC	0	90	40	Half Circle	Round Body
PPL10894	POLYPROPYLENE BU 0 X 90-30MM HC TC	0	90	30	Half Circle	Reverse Cutting
PPL200623	POLYPROPYLENE BU 2-0 X 90-60MM ST CT	2-0	90	60	Half Circle	Taper Cut
PPL200752	POLYPROPYLENE BU 2-0 X 90-75MM ST CT	2-0	90	75	Straight Needle	Cutting Needle
PPL200987	POLYPROPYLENE BU 2-0 X 90-90MM CU RC	2-0	90	90	Straight Needle	Cutting Needle
PPL208276	POLYPROPYLENE BU 2-0 X 100-75MM HC RB	2-0	100	75	3/8 Circle	Reverse Cutting
PPL20844	POLYPROPYLENE BU 2-0 X 90-30MM HC RB	2-0	90	30	Half Circle	Round Body
PPL208523	POLYPROPYLENE BU 2-0 X 90-26MM HC RB	2-0	90	26	Half Circle	Round Body
PPL208685	POLYPROPYLENE BU 2-0 X 75-24MM CU RC	2-0	75	24	Half Circle	Round Body
PPL208689	POLYPROPYLENE BU 2-0 X 90-40MM CU RC	2-0	90	40	3/8 Circle	Reverse Cutting
PPL208833	POLYPROPYLENE BU 2-0 X 75-26MM HC RB	2-0	75	26	3/8 Circle	Reverse Cutting
PPL208850	POLYPROPYLENE BU 2-0 X 90-30MM HC TC	2-0	90	30	Half Circle	Round Body
PPL208852	POLYPROPYLENE BU 2-0 X 90-40MM HC TC	2-0	90	40	Half Circle	Taper Cut
PPL208937	POLYPROPYLENE BU 2-0 X 90-17MM HC TC DN	2-0	90	17	Half Circle	Taper Cut
PPL208938	POLYPROPYLENE BU 2-0 X 90-17MM HC TC	2-0	90	17	Half Circle	Taper Cut, Double Needle
PPL208977	POLYPROPYLENE BU 2-0 X 90-26MM HC TC DN	2-0	90	26	Half Circle	Taper Cut
PPL208978	POLYPROPYLENE BU 2-0 X 75-26MM HC CT	2-0	75	26	Half Circle	Taper Cut, Double Needle
PPL30018	POLYPROPYLENE BU 3-0 X 70-22MM CU CT	3-0	70	22	Half Circle	Cutting Needle
PPL300621	POLYPROPYLENE BU 3-0 X 90-60MM ST CT	3-0	90	60	3/8 Circle	Cutting Needle
PPL300626	POLYPROPYLENE BU 3-0 X 90-35MM CU SM	3-0	90	35	Straight Needle	Cutting Needle

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL30800	POLYPROPYLENE BU 3-0 X 90-25MM CU CT	3-0	90	25	3/8 Circle	Spatulaed Musles Needle
PPL30825	POLYPROPYLENE BU 3-0 X 90-25MM HC RB	3-0	90	25	3/8 Circle	Cutting Needle
PPL30838	POLYPROPYLENE BU 3-0 X 70-25MM HC RB	3-0	70	25	Half Circle	Round Body
PPL308392	POLYPROPYLENE BU 3-0 X 90-20MM CU SM	3-0	90	20	Half Circle	Round Body
PPL308522	POLYPROPYLENE BU 3-0 X 90-26MM HC RB DN	3-0	90	26	3/8 Circle	Spatulaed Musles Needle
PPL308525	POLYPROPYLENE BU 3-0 X 90-30MM HC RB	3-0	90	30	Half Circle	Round Body, Double Needle
PPL308549	POLYPROPYLENE BU 3-0 X 90-16MM CU RC	3-0	90	16	Half Circle	Round Body
PPL308558	POLYPROPYLENE BU 3-0 X 90-17MM HC RB	3-0	90	17	3/8 Circle	Reverse Cutting
PPL308559	POLYPROPYLENE BU 3-0 X 90-16MM HC RB	3-0	90	16	Half Circle	Round Body
PPL308665	POLYPROPYLENE BU 3-0 X 45-19MM CU RC	3-0	45	19	Half Circle	Round Body
PPL308667	POLYPROPYLENE BU 3-0 X 90-16MM HC CT DN	3-0	90	16	3/8 Circle	Reverse Cutting
PPL308684	POLYPROPYLENE BU 3-0 X 45-24MM CU RC	3-0	45	24	Half Circle	Cutting Needle, Double Needle
PPL308685	POLYPROPYLENE BU 3-0 X 90-26MM CU RC	3-0	90	26	3/8 Circle	Reverse Cutting
PPL308687	POLYPROPYLENE BU 3-0 X 75-19MM CU RC	3-0	75	19	3/8 Circle	Reverse Cutting
PPL308731	POLYPROPYLENE BU 3-0 X 90-22MM HC RC	3-0	90	22	3/8 Circle	Reverse Cutting
PPL308770	POLYPROPYLENE BU 3-0 X 75-22MM HC RB	3-0	75	22	Half Circle	Reverse Cutting
PPL308832	POLYPROPYLENE BU 3-0 X 75-26MM HC RB	3-0	75	26	Half Circle	Round Body
PPL308833	POLYPROPYLENE BU 3-0 X 90-26MM HC RB TRO DN	3-0	90	26	Half Circle	Round Body
PPL308849	POLYPROPYLENE BU 3-0 X 90-30MM HC TC	3-0	90	30	Half Circle	Round Body, Double Needle, Trocar Point Needle
PPL30887	POLYPROPYLENE BU 3-0 X 70-20MM HC RB	3-0	70	20	Half Circle	Taper Cut
PPL30888	POLYPROPYLENE BU 3-0 X 70-60MM ST CT	3-0	70	60	Half Circle	Round Body
PPL308936	POLYPROPYLENE BU 3-0 X 90-17MM HC RB DN	3-0	90	17	Straight Needle	Cutting Needle
PPL308937	POLYPROPYLENE BU 3-0 X 90-17MM HC TC DN	3-0	90	17	Half Circle	Round Body, Double Needle
PPL308976	POLYPROPYLENE BU 3-0 X 90-26MM HC TC DN	3-0	90	26	Half Circle	Taper Cut, Double Needle
PPL308977	POLYPROPYLENE BU 3-0 X 90-25MM HC TC	3-0	90	25	Half Circle	Taper Cut, Double Needle
PPL309556	POLYPROPYLENE BU 3-0 X 90-20MM HC RB TRO DN	3-0	90	20	Half Circle	Taper Cut
PPL309557	POLYPROPYLENE BU 3-0 X 90-20MM HC RB TC DN	3-0	90	20	Half Circle	Round Body, Double Needle, Trocar Point Needle
PPL400525	POLYPROPYLENE BU 4-0 X 90-15MM CU SM	4-0	90	15	Half Circle	Taper Cut, Double Needle
PPL400539	POLYPROPYLENE BU 4-0 X 90-25MM CU SM	4-0	90	25	3/8 Circle	Spatulaed Musles Needle
PPL400835	POLYPROPYLENE BU 4-0 X 90-17MM HC TC	4-0	90	17	3/8 Circle	Spatulaed Musles Needle

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL408015	POLYPROPYLENE BU 4-0 X 90-19MM CU RC	4-0	90	19	Half Circle	Taper Cut
PPL408020	POLYPROPYLENE BU 4-0 X 90-26MM CU CT	4-0	90	26	3/8 Circle	Reverse Cutting
PPL408204	POLYPROPYLENE BU 4-0 X 90-13MM HC RB DN	4-0	90	13	3/8 Circle	Cutting Needle
PPL408355	POLYPROPYLENE BU 4-0 X 90-26MM HC RB	4-0	90	26	Half Circle	Round Body, Double Needle
PPL408391	POLYPROPYLENE BU 4-0 X 90-20MM CU SM	4-0	90	20	Half Circle	Round Body
PPL40849	POLYPROPYLENE BU 4-0 X 70-16MM HC RB	4-0	70	16	3/8 Circle	Spatulaed Musles Needle
PPL408521	POLYPROPYLENE BU 4-0 X 90-26MM HC RB DN	4-0	90	26	Half Circle	Round Body
PPL408557	POLYPROPYLENE BU 4-0 X 90-16MM HC RB DN	4-0	90	16	Half Circle	Round Body, Double Needle
PPL408558	POLYPROPYLENE BU 4-0 X 90-17MM HC RB	4-0	90	17	Half Circle	Round Body, Double Needle
PPL408634	POLYPROPYLENE BU 4-0 X 45-16MM CU SM	4-0	45	16	Half Circle	Round Body
PPL408665	POLYPROPYLENE BU 4-0 X 90-13MM HC TC	4-0	90	13	3/8 Circle	Spatulaed Musles Needle
PPL408666	POLYPROPYLENE BU 4-0 X 75-13MM HC CT DN	4-0	75	13	Half Circle	Taper Cut
PPL40867	POLYPROPYLENE BU 4-0 X 90-25MM HC TC DN	4-0	90	25	Half Circle	Cutting Needle, Double Needle
PPL408682	POLYPROPYLENE BU 4-0 X 45-19MM CU RC	4-0	45	19	Half Circle	Taper Cut, Double Needle
PPL408683	POLYPROPYLENE BU 4-0 X 45-20MM CU RC	4-0	45	20	3/8 Circle	Reverse Cutting
PPL40870	POLYPROPYLENE BU 4-0 X 70-16MM CU CT	4-0	70	16	3/8 Circle	Reverse Cutting
PPL408761	POLYPROPYLENE BU 4-0 X 90-20MM HC RB	4-0	90	20	3/8 Circle	Cutting Needle
PPL408840	POLYPROPYLENE BU 4-0 X 90-20MM HC CT DN	4-0	90	20	Half Circle	Round Body
PPL408841	POLYPROPYLENE BU 4-0 X 90-20MM HC TC	4-0	90	20	Half Circle	Cutting Needle, Double Needle
PPL408845	POLYPROPYLENE BU 4-0 X 90-25MM HC TC DN	4-0	90	25	Half Circle	Taper Cut
PPL408846	POLYPROPYLENE BU 4-0 X 90-26MM HC CT DN	4-0	90	26	Half Circle	Taper Cut, Double Needle
PPL408861	POLYPROPYLENE BU 4-0 X 90-18MM CU RB DN	4-0	90	18	Half Circle	Cutting Needle, Double Needle
PPL408875	POLYPROPYLENE BU 4-0 X 90-16MM CU CT	4-0	90	16	3/8 Circle	Round Body, Double Needle
PPL408879	POLYPROPYLENE BU 4-0 X 90-25MM CU CT	4-0	90	25	3/8 Circle	Cutting Needle
PPL408935	POLYPROPYLENE BU 4-0 X 90-17MM HC TC DN	4-0	90	17	3/8 Circle	Cutting Needle
PPL500527	POLYPROPYLENE BU 5-0 X 90-15MM CU SM	5-0	90	15	Half Circle	Taper Cut, Double Needle
PPL508006	POLYPROPYLENE BU 5-0 X 90-16MM CU CT	5-0	90	16	3/8 Circle	Spatulaed Musles Needle
PPL508010	POLYPROPYLENE BU 5-0 X 90-19MM CU CT	5-0	90	19	3/8 Circle	Cutting Needle
PPL508014	POLYPROPYLENE BU 5-0 X 90-19MM CU RC	5-0	90	19	3/8 Circle	Cutting Needle
PPL50810	POLYPROPYLENE BU 5-0 X 75-13MM HC RB DN TP	5-0	75	13	3/8 Circle	Reverse Cutting
PPL508205	POLYPROPYLENE BU 5-0 X 60-13MM HC RB DN	5-0	60	13	Half Circle	Round Body, Double Needle
PPL508556	POLYPROPYLENE BU 5-0 X 90-16MM HC RB DN	5-0	90	16	Half Circle	Round Body, Double Needle

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL508557	POLYPROPYLENE BU 5-0 X 90-17MM HC RB	5-0	90	17	Half Circle	Round Body
PPL50856	POLYPROPYLENE BU 5-0 X 90-16MM HCRB DN	5-0	90	16	Half Circle	Round Body, Double Needle
PPL508662	POLYPROPYLENE BU 5-0 X 90-08MM CU TC	5-0	90	8	3/8 Circle	Taper Cut
PPL508663	POLYPROPYLENE BU 5-0 X 60-09MM CU CT DN	5-0	60	9	3/8 Circle	Cutting Needle, Double Needle
PPL508664	POLYPROPYLENE BU 5-0 X 60-13MM HC CT DN	5-0	60	13	Half Circle	Cutting Needle, Double Needle
PPL508698	POLYPROPYLENE BU 5-0 X 45-13MM CU RC	5-0	45	13	3/8 Circle	Reverse Cutting
PPL508710	POLYPROPYLENE BU 5-0 X 75-13MM HC RB DN	5-0	75	13	Half Circle	Round Body, Double Needle
PPL508711	POLYPROPYLENE BU 5-0 X 75-13MM HC RB	5-0	75	13	Half Circle	Round Body
PPL508720	POLYPROPYLENE BU 5-0 X 75-13MM CU CT DN	5-0	75	13	3/8 Circle	Cutting Needle, Double Needle
PPL508721	POLYPROPYLENE BU 5-0 X 75-13MM CU RB DN	5-0	75	13	3/8 Circle	Round Body, Double Needle
PPL508722	POLYPROPYLENE BU 5-0 X 90-13MM CU TC	5-0	90	13	3/8 Circle	Taper Cut
PPL508756	POLYPROPYLENE BU 5-0 X 90-17MM HC RB DN	5-0	90	17	Half Circle	Round Body, Double Needle
PPL508757	POLYPROPYLENE BU 5-0 X 90-16MM HC RB TC DN	5-0	90	16	Half Circle	Round Body, Taper Cut, Double Needle
PPL508802BX	POLYPROPYLENE BU 5-0 X 75-11MM HC RB DVB	5-0	75	11	Half Circle	Round Body, Double Needle, Visible Black
PPL508803	POLYPROPYLENE BU 5-0 X 60-10MM HC RB DN	5-0	60	10	Half Circle	Round Body, Double Needle
PPL508804	POLYPROPYLENE BU 5-0 X 60-11MM HC CT DN	5-0	60	11	Half Circle	Cutting Needle, Double Needle
PPL50881	POLYPROPYLENE BU 5-0 X 70-16MM CU RB	5-0	70	16	3/8 Circle	Round Body
PPL508816	POLYPROPYLENE BU 5-0 X 75-13MM HC CT DN	5-0	75	13	Half Circle	Cutting Needle, Double Needle
PPL50882DN	POLYPROPYLENE BU 5-0X 70-16MM CU RB DN	5-0	70	16	3/8 Circle	Round Body, Double Needle
PPL508881	POLYPROPYLENE BU 5-0 X 90-25MM CU CT	5-0	90	25	3/8 Circle	Cutting Needle
PPL508882	POLYPROPYLENE BU 5-0 X 90-16MM CU RC	5-0	90	16	3/8 Circle	Reverse Cutting
PPL50889	POLYPROPYLENE BU 5-0 X 90-12MM CU RC	5-0	90	12	3/8 Circle	Reverse Cutting
PPL508890	POLYPROPYLENE BU 5-0 X 75-13MM CU RB	5-0	75	13	3/8 Circle	Round Body
PPL600523	POLYPROPYLENE BU 6-0 X 90-15MM CU SM	6-0	90	15	3/8 Circle	Spatulaed Musles Needle
PPL608000	POLYPROPYLENE BU 6-0 X 90-09MM CU RC	6-0	90	9	3/8 Circle	Reverse Cutting
PPL60818	POLYPROPYLENE BU 6-0 X 60-13MM CU RB DN	6-0	60	13	3/8 Circle	Round Body, Double Needle
PPL60823	POLYPROPYLENE BU 6-0 X 70-15MM CU CT	6-0	70	15	3/8 Circle	Cutting Needle
PPL60829	POLYPROPYLENE BU 6-0 X 70-13MM CU RB DN	6-0	70	13	3/8 Circle	Round Body, Double Needle
PPL608307BL	POLYPROPYLENE BU 6-0 X 60-13MM CU RB DN VB	6-0	60	13	3/8 Circle	Round Body, Double Needle, V-Black
PPL608597	POLYPROPYLENE BU 6-0 X 60-11MM CU RB DN	6-0	60	11	3/8 Circle	Round Body, Double Needle



## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL608617	POLYPROPYLENE BU 6-0 X 45-13MM CU SM	6-0	45	13	3/8 Circle	Spatulaed Musles Needle
PPL608697	POLYPROPYLENE BU 6-0 X 45-11MM CU SM	6-0	45	11	3/8 Circle	Spatulaed Musles Needle
PPL608706	POLYPROPYLENE BU 6-0 X 75-13MM CU RB DN V	6-0	75	13	3/8 Circle	Cutting Needle, Double Needle, Visible Black
PPL608707	POLYPROPYLENE BU 6-0 X 60-13MM CU CT DN	6-0	60	13	3/8 Circle	Cutting Needle, Double Needle
PPL608708	POLYPROPYLENE BU 6-0 X 90-13MM CU CT	6-0	90	13	3/8 Circle	Taper Cut
PPL608714	POLYPROPYLENE BU 6-0 X 75-13MM HC RB DN	6-0	75	13	Half Circle	Round Body, Double Needle
PPL608718	POLYPROPYLENE BU 6-0 X 60-13MM CU RB DN TH	6-0	60	13	3/8 Circle	Round Body, Double Needle, Thick
PPL60872	POLYPROPYLENE BU 6-0 X 60-9MM CU RB DN	6-0	60	9	3/8 Circle	Round Body, Double Needle
PPL608726	POLYPROPYLENE BU 6-0 X 60-13MM CURB DN TN	6-0	60	13	3/8 Circle	Round Body, Double Needle, Thin
PPL608726BL	POLYPROPYLENE BU 6-0 X 60-13MM CU RB DVB	6-0	60	13	3/8 Circle	Round Body, Double Needle, Visible Black
PPL608726BX	POLYPROPYLENE BU 6-0 X 75-13MM CU RB D VB	6-0	75	13	3/8 Circle	Round Body, Double Needle, Visible Black
PPL608726XL	POLYPROPYLENE BU 6-0 X 75-13MM CU RB DN	6-0	75	13	3/8 Circle	Round Body, Double Needle
PPL608776	POLYPROPYLENE BU 6-0 X 75-12MM CU RB DN	6-0	75	12	3/8 Circle	Round Body, Double Needle
PPL60878	POLYPROPYLENE BU 6-0 X 45-12MM CU RC	6-0	45	12	3/8 Circle	Reverse Cutting
PPL608802	POLYPROPYLENE BU 6-0 X 60-11MM CU CT DN	6-0	60	11	3/8 Circle	Cutting Needle, Double Needle
PPL608803	POLYPROPYLENE BU 6-0 X 90-11MM CU CT	6-0	90	11	3/8 Circle	Taper Cut
PPL608804	POLYPROPYLENE BU 6-0 X 60-10MM CU RB DN	6-0	60	10	3/8 Circle	Round Body, Double Needle
PPL608804BX	POLYPROPYLENE BU 6-0 X 60-10MM CU RB DN VB	6-0	60	10	3/8 Circle	Round Body, Double Needle, Visible Black
PPL608805	POLYPROPYLENE BU 6-0 X 60-9MM CU RB DN	6-0	60	9	3/8 Circle	Round Body, Double Needle
PPL608806	POLYPROPYLENE BU 6-0 X 90-09MM CU CT	6-0	90	9	3/8 Circle	Taper Cut
PPL608807	POLYPROPYLENE BU 6-0 X 60-09MM CU CT DN	6-0	60	9	3/8 Circle	Cutting Needle, Double Needle
PPL608814	POLYPROPYLENE BU 6-0 X 75-13MM CU CT DN	6-0	75	13	3/8 Circle	Cutting Needle, Double Needle
PPL608815	POLYPROPYLENE BU 6-0 X 60-13MM HC CT DN	6-0	60	13	Half Circle	Cutting Needle, Double Needle
PPL608873	POLYPROPYLENE BU 6-0 X 90-16MM CU CT	6-0	90	16	3/8 Circle	Cutting Needle
PPL60893	POLYPROPYLENE BU 6-0 X 70-8MM CU RB DN	6-0	70	8	3/8 Circle	Round Body, Double Needle
PPL60895DN	POLYPROPYLENE BU 6-0 X 70-10MM CU RB DN	6-0	70	10	3/8 Circle	Round Body, Double Needle
PPL70804	POLYPROPYLENE BU 7-0 X 60-8MM CU CT DN	7-0	60	8	3/8 Circle	Cutting Needle, Double Needle
PPL708070	POLYPROPYLENE BU 7-0 X 60-10MM CU RB DN	7-0	60	10	3/8 Circle	Round Body, Double Needle

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL708078	POLYPROPYLENE BU 7-0 X 60-8MM CU RB DN	7-0	60	8	3/8 Circle	Round Body, Double Needle
PPL708078BL	POLYPROPYLENE BU 7-0 X 60-8MM CU RB DN VB	7-0	60	8	3/8 Circle	Round Body, Double Needle, Visible Black
PPL70811	POLYPROPYLENE BU 7-0 X 60-6MM CU RB DN	7-0	60	6	3/8 Circle	Round Body, Double Needle
PPL708304	POLYPROPYLENE BU 7-0 X 60-09MM CU RB DN	7-0	60	9	3/8 Circle	Round Body, Double Needle
PPL70866	POLYPROPYLENE BU 7-0 X 60-8MM CU RB DN	7-0	60	8	3/8 Circle	Round Body, Double Needle
PPL70866ML	POLYPROPYLENE BU 2 X 7-0X60-8MM CU RB DN	2	60	8	3/8 Circle	Round Body, Double Needle
PPL708701	POLYPROPYLENE BU 7-0 X 45-9MM CU RB DN	7-0	45	9	3/8 Circle	Round Body, Double Needle
PPL708702	POLYPROPYLENE BU 7-0 X 60-9MM CU RB DN	7-0	60	9	3/8 Circle	Round Body, Double Needle
PPL708704	POLYPROPYLENE BU 7-0 X 60-9MM CU CT DN	7-0	60	9	3/8 Circle	Cutting Needle, Double Needle
PPL708705	POLYPROPYLENE BU 7-0 X 60-09MM CU CT DN	7-0	60	9	3/8 Circle	Cutting Needle, Double Needle
PPL708725	POLYPROPYLENE BU 7-0 X 60-13MM CU RB DN	7-0	60	13	3/8 Circle	Round Body, Double Needle
PPL70873	POLYPROPYLENE BU 7-0 X 60-9MM CU RB DN	7-0	60	9	3/8 Circle	Round Body, Double Needle
PPL708735	POLYPROPYLENE BU 7-0 X 60-8MM CU RB DN	7-0	60	8	3/8 Circle	Round Body, Double Needle
PPL708801	POLYPROPYLENE BU 7-0 X 90-11MM CU CT	7-0	90	11	3/8 Circle	Cutting Needle
PPL708802	POLYPROPYLENE BU 7-0 X 60-11MM CU CT DN	7-0	60	11	3/8 Circle	Cutting Needle, Double Needle
PPL708813	POLYPROPYLENE BU 7-0 X 60-13MM CU CT DN	7-0	60	13	3/8 Circle	Cutting Needle, Double Needle
PPL802774	POLYPROPYLENE BU 8-0 X 75-06MM CU RB	8-0	75	6	3/8 Circle	Round Body
PPL802775	POLYPROPYLENE BU 8-0 X 13-06MM CU RB	8-0	13	6	3/8 Circle	Round Body
PPL802777	POLYPROPYLENE BU 8-0 X 45-06MM CU RB DN	8-0	45	6	3/8 Circle	Round Body, Double Needle
PPL808086XF	POLYPROPYLENE BU 8-0 X 60-6MM CU RB DN TH	8-0	60	6	3/8 Circle	Round Body, Double Needle, Thick
PPL808088	POLYPROPYLENE BU 8-0 X 60-8MM CU RB DN	8-0	60	8	3/8 Circle	Round Body, Double Needle
PPL808101BL	POLYPROPYLENE BU 8-0 X 60-8MM CU CT VB D	8-0	60	8	3/8 Circle	Cutting Needle, Double Needle, Visible Black
PPL80812	POLYPROPYLENE BU 8-0 X 60-6MM CU RB DN TN	8-0	60	6	3/8 Circle	Round Body, Double Needle, Thin
PPL80813	POLYPROPYLENE BU 8-0 X 60-9MM CU RB DN	8-0	60	9	3/8 Circle	Round Body, Double Needle
PPL808703	POLYPROPYLENE BU 8-0 X 60-09MM CU CT DN V	8-0	60	9	3/8 Circle	Cutting Needle, Double Needle, Visible Black
PPL808704	POLYPROPYLENE BU 8-0 X 90-8.8MM CU CT DN	8-0	90	8,8	3/8 Circle	Cutting Needle, Double Needle
PPL808732	POLYPROPYLENE BU 8-0 X 60-6MM CU RB DN	8-0	60	6	3/8 Circle	Round Body, Double Needle
PPL808741	POLYPROPYLENE BU 8-0 X 60-8MM CU RB DN TH	8-0	60	8	3/8 Circle	Round Body, Double Needle, Thick



## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL80876	POLYPROPYLENE BU 8-0 X 60-8MM CU RB DN TN	8-0	60	8	3/8 Circle	Round Body, Double Needle, Thin
PPL80876ML	POLYPROPYLENE BU 2 X 8-0X60-8MM CU RB DN	2	60	8	3/8 Circle	Round Body, Double Needle
PPL902780	POLYPROPYLENE BU 8-0 X 90-4.8MM CU RB DN	8-0	90	4,8	3/8 Circle	Round Body, Double Needle
PPL108026	POLYPROPYLENE BU 0 X 75-26MM HC RB	0	75	26	Half Circle	Round Body
PPL20844SL	POLYPROPYLENE BU 2-0 X 75-30MM HC RB	2-0	75	30	Half Circle	Round Body
PPL20807	POLYPROPYLENE BU 2-0 X 70-25MM HC TC	2-0	70	25	Half Circle	Taper Cut
PPL308849DN	POLYPROPYLENE BU 3-0 X 90-31MM HC TC DN	3-0	90	31	Half Circle	Taper Cut, Double Needle
PPL408417	POLYPROPYLENE BU 4-0 X 90-16MM HC TC DN	4-0	90	16	Half Circle	Taper Cut, Double Needle
PPL408841DN	POLYPROPYLENE BU 4-0 X 90-20MM HC TC DN	4-0	90	20	Half Circle	Taper Cut, Double Needle
PPL408425	POLYPROPYLENE BU 4-0 X90-25MM HC TC	4-0	90	25	Half Circle	Taper Cut
PPL508830	POLYPROPYLENE BU 5-0 X 90-16MM HC TC DN	5-0	90	16	Half Circle	Taper Cut, Double Needle
PPL608889	POLYPROPYLENE BU 6-0 X 75-13MM CU RB	6-0	75	13	3/8 Circle	Round Body
PPL60817	POLYPROPYLENE BU 6-0 X 70-13MM CU CT	6-0	70	13	3/8 Circle	Cutting Needle
PPL208236	POLYPROPYLENE BU 2-0 X 90-36MM HC RB	2-0	90	36	Half Circle	Round Body
PPL1001713E	POLYPROPYLENE BU 10-0 X 45-06MM CU SM	10-0	45	6	3/8 Circle	Spatulaed Musles Needle
PPL508686	POLYPROPYLENE BU 5-0 X 45-19MM CU RC	5-0	45	19	3/8 Circle	Reverse Cutting
PPL6013433	POLYPROPYLENE BU 6-0 X 45-10MM CU RC	6-0	45	10	3/8 Circle	Reverse Cutting
PPL108026Y	POLYPROPYLENE BU 0 X 75-26MM HC RB-2D	0	75	26	Half Circle	Round Body
PPL20844SLY	POLYPROPYLENE BU 2-0 X 75-30MM HC RB-2D	2-0	75	30	Half Circle	Round Body
PPL308832Y	POLYPROPYLENE BU 3-0 X 75-26MM HC RB-2D	3-0	75	26	Half Circle	Round Body
PPL408355Y	POLYPROPYLENE BU 4-0 X 90-26MM HC RB-2D	4-0	90	26	Half Circle	Round Body
PPL20807Y	POLYPROPYLENE BU 2-0 X 70-25MM HC TC-2D	2-0	70	25	Half Circle	Taper Cut
PPL308667Y	POLYPROPYLENE BU 3-0 X 90-16MM HC TC DN-2D	3-0	90	16	Half Circle	Taper Cut, Double Needle
PPL308977Y	POLYPROPYLENE BU 3-0 X 90-25MM HC TC-2D	3-0	90	25	Half Circle	Taper Cut
PPL308849DNY	POLYPROPYLENE BU 3-0 X 90-31MM HC TC DN-2D	3-0	90	31	Half Circle	Taper Cut, Double Needle
PPL408417Y	POLYPROPYLENE BU 4-0 X 90-16MM HC TC DN-2D	4-0	90	16	Half Circle	Taper Cut, Double Needle
PPL40849Y	POLYPROPYLENE BU 4-0 X 70-16MM HC RB-2D	4-0	70	16	Half Circle	Round Body
PPL408841DNY	POLYPROPYLENE BU 4-0 X 90-20MM HC TC DN-2D	4-0	90	20	Half Circle	Taper Cut, Double Needle
PPL408425Y	POLYPROPYLENE BU 4-0 X90-25MM HC TC-2D	4-0	90	25	Half Circle	Taper Cut
PPL508804Y	POLYPROPYLENE BU 5-0 X 60-11MM HC TC DN-2D	5-0	60	11	Half Circle	Taper Cut, Double Needle
PPL508830Y	POLYPROPYLENE BU 5-0 X 90-16MM HC TC DN-2D	5-0	90	16	Half Circle	Taper Cut, Double Needle

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL608889Y	POLYPROPYLENE BU 6-0 X 75-13MM CU RB-2D	6-0	75	13	3/8 Circle	Round Body
PPL60817Y	POLYPROPYLENE BU 6-0 X 70-13MM CU CT-2D	6-0	70	13	3/8 Circle	Cutting Needle
PPL708070Y	POLYPROPYLENE BU 7-0 X 60-10MM CU RB DN-2D	7-0	60	10	3/8 Circle	Round Body, Double Needle
PPL708813Y	POLYPROPYLENE BU 7-0 X 60-13MM CU TC DN-2D	7-0	60	13	3/8 Circle	Taper Cut, Double Needle
PPL708725Y	POLYPROPYLENE BU 7-0 X 60-13MM CU RB DN-2D	7-0	60	13	3/8 Circle	Round Body, Double Needle
PPL708704Y	POLYPROPYLENE BU 7-0 X 60-9MM CU TC DN-2D	7-0	60	9	3/8 Circle	Taper Cut, Double Needle
PPL108424Y	POLYPROPYLENE BU 0 X 75-37MM HC RB-2D	0	75	37	Half Circle	Round Body
PPL108434Y	POLYPROPYLENE BU 0 X 75-40MM HC RB-2D	0	75	40	Half Circle	Round Body
PPL208833Y	POLYPROPYLENE BU 2-0 X 75-26MM HC RB-2D	2-0	75	26	Half Circle	Round Body
PPL208236Y	POLYPROPYLENE BU 2-0 X 90-36MM HC RB-2D	2-0	90	36	Half Circle	Round Body
PPL208685Y	POLYPROPYLENE BU 2-0 X 75-24MM CU RC-2D	2-0	75	24	3/8 Circle	Reverse Cutting
PPL308684Y	POLYPROPYLENE BU 3-0 X 45-24MM CU RC-2D	3-0	45	24	3/8 Circle	Reverse Cutting
PPL408682Y	POLYPROPYLENE BU 4-0 X 45-19MM CU RC-2D	4-0	45	19	3/8 Circle	Reverse Cutting
PPL508686Y	POLYPROPYLENE BU 5-0 X 45-19MM CU RC-2D	5-0	45	19	3/8 Circle	Reverse Cutting
PPL6013433Y	POLYPROPYLENE BU 6-0 X 45-10MM CU RC-2D	6-0	45	10	3/8 Circle	Reverse Cutting,
PPL808101BLZ	POLYPROPYLENE BU 8-0 X 60-8MM CU TC VB DN-3D	8-0	60	8	3/8 Circle	Taper Cut, Visible Black, Double Needle
PPL608726	POLYPROPYLENE BU 6-0 X 60-13MM CU RB DN TN	6-0	60	13	3/8 Circle	Round Body, Double Needle, FINO
PPL01884	POLYPROPYLENE BU 1 X 100-40MM HC RC	1	100	40	Half Circle	Reverse Cutting
PPL01842	POLYPROPYLENE BU 1 X 100-36MM HC RB	1	100	36	Half Circle	Round Body
PPL408684	POLYPROPYLENE BU 4-0 X 90-24MM CU RC	4-0	90	24	3/8 Circle	Reverse Cutting
PPL208685XL	POLYPROPYLENE BU 2-0 X 100-24MM CU RC	2-0	100	24	3/8 Circle	Reverse Cutting
PPL408682XL	POLYPROPYLENE BU 4-0 X 70-19MM CU RC	4-0	70	19	3/8 Circle	Reverse Cutting
PPL208834	POLYPROPYLENE BU 2-0 X 75-26MM HC RB DN	2-0	75	26	Half Circle	Round Body, Double Needle
PPL108024	POLYPROPYLENE BU 0 X 75-24MM CU CT	0	75	24	3/8 Circle	Cutting Needle
PPL018190	POLYPROPYLENE BU 1 X 90-90MM CU RC	1	90	90	3/8 Circle	Reverse Cutting
PPL200753	POLYPROPYLENE BU 2-0 X 90-70MM ST CT	2-0	90	70	Straight Needle	Cutting Needle
PPL308317	POLYPROPYLENE BU 3-0 X 90-17MM HC TC	3-0	90	17	Half Circle	Taper Cut
PPL308336	POLYPROPYLENE BU 3-0 X 90-36MM HC RB	3-0	90	36	Half Circle	Round Body
PPL408416	POLYPROPYLENE BU 4-0 X 90-16MM CU RC	4-0	90	16	3/8 Circle	Reverse Cutting
PPL508526	POLYPROPYLENE BU 5-0 X 75-26MM HC RB DN	5-0	75	26	Half Circle	Round Body, Double Needle
PPL508517	POLYPROPYLENE BU 5-0 X 75-17MM HC TC DN	5-0	75	17	Half Circle	Taper Cut, Double Needle

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL308771	POLYPROPYLENE BU 3-0 X 75-22MM HC RB DN	3-0	75	22	Half Circle	Round Body, Double Needle
PPL608610	POLYPROPYLENE BU 6-0 X 75-10MM HC RB DN	6-0	75	10	Half Circle	Round Body, Double Needle
PPL608616	POLYPROPYLENE BU 6-0 X 75-16MM CU RC	6-0	75	16	3/8 Circle	Reverse Cutting
PPL20295	POLYPROPYLENE BU 2-0 X 90-31MM HC RB VB	2-0	90	31	Half Circle	Round Body, Visible Black
PPL028245	POLYPROPYLENE BU 2 X 75-45MM HC RC	2	75	45	Half Circle	Reverse Cutting
PPL028275	POLYPROPYLENE BU 2 X 75-45MM HC RB	2	75	45	Half Circle	Round Body
PPL108045	POLYPROPYLENE BU 0 X75-45MM HC RC	0	75	45	Half Circle	Reverse Cutting
PPL108075	POLYPROPYLENE BU 0 X75-45MM HC RB	0	75	45	Half Circle	Round Body
PPL208245	POLYPROPYLENE BU 2-0X 75-45MM HC RC	2-0	75	45	Half Circle	Reverse Cutting
PPL208275	POLYPROPYLENE BU 2-0X 75-45MM HC RB	2-0	75	45	Half Circle	Round Body
PPL018130	POLYPROPYLENE BU 1 X 75-30MM HC RB TRO DN	1	75	30	Half Circle	Round Body, Trocar Point Needle, Double Needle
PPL208230	POLYPROPYLENE BU 2-0X 70-30MM CU RB	2-0	70	30	3/8 Circle	Round Body
PPL01843DS	POLYPROPYLENE BU 1 X100-40MM HC RB	1	100	40	Half Circle	Round Body
PPL10824MIPH	POLYPROPYLENE BU 0 X90-26MM HC RB DN	0	90	26	Half Circle	Round Body, Double Needle
PPL408526DN	POLYPROPYLENE BU 4-0X 75-22MM HC RB DN	4-0	75	22	Half Circle	Round Body, Double Needle
PPL1001713DN	POLYPROPYLENE BU 10-0X 23-16MM ST CT DN	10-0	23	16	Straight	Cutting Needle, Double Needle
PPL708304BL	POLYPROPYLENE BU 7-0 X 60-9MM CURB DN VB	7-0	60	9	3/8 Circle	Round Body, Double Needle, Visible Black
PPL40835	POLYPROPYLENE BU 4-0X 70-17MM HC TC	4-0	70	17	Half Circle	Taper Cut
PPL50826	POLYPROPYLENE BU 5-0X 70-15MM CU CT	5-0	70	15	3/8 Circle	Cutting Needle
PPL608860	POLYPROPYLENE BU 6-0X 60-13MM HC RB DVB	6-0	60	13	Half Circle	Round Body,
PPL3013413	POLYPROPYLENE BU 3-0X 75-60MM ST CT	3-0	75	60	Straight	Cutting Needle
PPL3013453	POLYPROPYLENE BU 3-0 X 75-16MM CU CT	3-0	75	16	3/8 Circle	Cutting Needle
PPL2085200	POLYPROPYLENE BU 2-0 X 75-20MM HC RB	2-0	75	20	Half Circle	Round Body
PPL408884	POLYPROPYLENE BU 4-0 X 45-16MM CU RC	4-0	45	16	3/8 Circle	Reverse Cutting
PPL308534	POLYPROPYLENE BU 3-0 X 120-26MM HC RB	3-0	120	26	Half Circle	Round Body
PPL505969	POLYPROPYLENE BU 5-0 X 90-12MM CU TC DN	5-0	90	12	3/8 Circle	Taper Cut, Double Needle
PPL307694	POLYPROPYLENE BU 3-0 X 75-24MM CU RC	3-0	75	24	3/8 Circle	Reverse Cutting
PPL307499	POLYPROPYLENE BU 3-0 X 90-36MM HC RB DN	3-0	90	36	Half Circle	Round Body, Double Needle
PPL207697	POLYPROPYLENE BU 2-0 X 75-30MM CU RC	2-0	75	30	3/8 Circle	Reverse Cutting
PPL607975	POLYPROPYLENE BU 6-0 X 75-13MM CU TC	6-0	75	13	3/8 Circle	Taper Cut
PPL607406	POLYPROPYLENE BU 6-0 X 75-9.3MM CU RB DN	6-0	75	9.3	3/8 Circle	Round Body, Double Needle

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL508717	POLYPROPYLENE BU 5-0 X 45-13MM CU RB DN	5-0	45	13	3/8 Circle	Round Body, Double Needle
PPL307778	POLYPROPYLENE BU 3-0 X 120-26MM HC TC DN	3-0	120	26	Half Circle	Taper Cut, Double Needle
PPL200631	POLYPROPYLENE BU 2-0 X 75-60MM ST CT	2-0	75	60	Straight Needle	Cutting Needle
PPL208231	POLYPROPYLENE BU 2-0 X 75-31MM HC RB	2-0	75	31	Half Circle	Round Body
PPL608005	POLYPROPYLENE BU 6-0 X 45-16MM CU TC	6-0	45	16	3/8 Circle	Taper Cut
PPL508006SL	POLYPROPYLENE BU 5-0X 45-16MM CU CT	5-0	45	16	3/8 Circle	Cutting Needle
PPL408007	POLYPROPYLENE BU 4-0X 45-16MM CU CT	4-0	45	16	3/8 Circle	Cutting Needle
PPL408020SL	POLYPROPYLENE BU 4-0X 45-26MM CU CT	4-0	45	26	3/8 Circle	Cutting Needle
PPL308025	POLYPROPYLENE BU 3-0X 45-26MM CU RC	3-0	45	26	3/8 Circle	ReverseCutting
PPL108434XL	POLYPROPYLENE BU 0 X100-40MM HC RB	0	100	40	Half Circle	Round Body
PPL208526	POLYPROPYLENE BU 2-0 X 90-31MM HC RB DN	2-0	90	31	Half Circle	Round Body, Double Needle
PPL608868	POLYPROPYLENE BU 6-0X 45-11MM CU RC	6-0	45	11	3/8 Circle	Reverse Cutting
PPL508882SL	POLYPROPYLENE BU 5-0X 45-16MM CU RC	5-0	45	16	3/8 Circle	Reverse Cutting
PPL018999	POLYPROPYLENE BU 1 X100-40MM HC RB BP	1	100	40	Half Circle	Round Body, Blunt Point
PPL10982J	POLYPROPYLENE BU 0 X100-31MM J TYPE TC	0	100	31	J TYPE	Taper Cut
PPL1002794	POLYPROPYLENE BU 10-0X 13-5MM CU RB DN	10-0	13	5	3/8 Circle	Round Body, Double Needle
PPL408011	POLYPROPYLENE BU 4-0X 45-19MM CU CT	4-0	45	19	3/8 Circle	Cutting Needle
PPL208026	POLYPROPYLENE BU 2-0X 45-26MM CU RC	2-0	45	26	3/8 Circle	Reverse
PPL308525DN	POLYPROPYLENE BU 3-0 X 90-31MM HC RB DN	3-0	90	31	Half Circle	Round Body, Double Needle
PPL608712	POLYPROPYLENE BU 6-0 X 60-9.3MM CU RB DN	6-0	60	9.3	3/8 Circle	Round Body, Double Needle
PPL208852DN	POLYPROPYLENE BU 2-0 X 90-40MM HC TC DN	2-0	90	40	Half Circle	Taper Cut, Double Needle
PPL018140	POLYPROPYLENE BU 1 X 75-40MM CU RC	1	75	40	3/8 Circle	Reverse Cutting
PPL808741XL	POLYPROPYLENE BU 8-0 X 75-8MM CU RB DN TH	8-0	75	8	3/8 Circle	Round Body, Double Needle, Thick
PPL708078BXL	POLYPROPYLENE BU 7-0 X 75-8MM CU RB DN VB	7-0	75	8	3/8 Circle	Round Body, Double Needle, Visible Black
PPL40857	POLYPROPYLENE BU 4-0 X 90-17MM HC TC	4-0	90	17	Half Circle	Taper Cut
PPL40879	POLYPROPYLENE BU 4-0 X 45-12MM CU RC	4-0	45	12	3/8 Circle	Reverse Cutting
PPL50827DN	POLYPROPYLENE BU 5-0 X 70-12MM CU RB DN	5-0	70	12	3/8 Circle	Round Body, Double Needle
PPL50862	POLYPROPYLENE BU 5-0 X 70-13MM CU RC	5-0	70	13	3/8 Circle	Reverse Cutting
PPL708700	POLYPROPYLENE BU 7-0 X 60-11MM CU TC DN	7-0	60	11	3/8 Circle	Taper Cut, Double Deedle
PPL708021	POLYPROPYLENE BU 7-0 X 60-9MM CU RB DN VB	7-0	60	9	3/8 Circle	Round Body, Double Needle, Visible Black

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL608305	POLYPROPYLENE BU 6-0 X60-9MM CU RB DN VB	6-0	60	9	3/8 Circle	Round Body, Double Needle, Visible Black
PPL308878	POLYPROPYLENE BU 3-0 X 45-19MM CU CT	3-0	45	19	3/8 Circle	Cutting Needle
PPL308355	POLYPROPYLENE BU 3-0 X 90-26MM HC RB DN V	3-0	90	26	Half Circle	Round Body, Double Needle, Visible Black
PPL108430	POLYPROPYLENE BU 0 X 100-31MM HC RB	0	100	31	Half Circle	Round Body
PPL208571	POLYPROPYLENE BU 2-0 X 90-55MM ST CT	2-0	90	55	Straight Needle	Cutting Needle
PPL308851	POLYPROPYLENE BU 3-0 X 90-40MM HC TC DN	3-0	90	40	Half Circle	Taper Cut, Double Needle
PPL608871	POLYPROPYLENE BU 6-0 X 45-13MM CU RC	6-0	45	13	3/8 Circle	Reverse Cutting
PPL608802XL	POLYPROPYLENE BU 6-0 X 75-11MM CU TC DN	6-0	75	11	3/8 Circle	Taper Cut, Double Needle
PPL708704XL	POLYPROPYLENE BU 7-0 X 75-9MM CU TC DN	7-0	75	9	3/8 Circle	Taper Cut, Double Needle
PPL808732XL	POLYPROPYLENE BU 8-0 X 75-6.5MM CU RB DN	8-0	75	6.5	3/8 Circle	Round Body, Double Needle
PPL707402	POLYPROPYLENE BU 7-0 X 75-9MM CU RB	7-0	75	9	3/8 Circle	Round Body
PPL708701XL	POLYPROPYLENE BU 7-0 X 75-9MM CU RB DN	7-0	75	9	3/8 Circle	Round Body, Double Needle
PPL902783	POLYPROPYLENE BU 9-0 X 13-5.1MM CU RB DN	9-0	13	5.1	3/8 Circle	Round Body, Double Needle
PPL608307	POLYPROPYLENE BU 6-0 X60-13MM CU RB DN VB	6-0	60	13	3/8 Circle	Round Body, Double Needle, Visible Black
PPL508321	POLYPROPYLENE BU 5-0 X75-13MM CU RB DN VB	5-0	75	13	3/8 Circle	Round Body, Double Needle, Visible Black
PPL707222	POLYPROPYLENE BU 7-0 X 75-9MM CU RB DN VB	7-0	75	9	3/8 Circle	Round Body, Double Needle, Visible Black
PPL707225	POLYPROPYLENE BU 7-0 X 45-9MM CU RB DN VB	7-0	45	9	3/8 Circle	Round Body, Double Needle, Visible Black
PPL40899	POLYPROPYLENE BU 4-0 X 70-17MM HC TC	4-0	70	17	Half Circle	Taper Cut
PPL10842	POLYPROPYLENE BU 0 X 70-30MM HC RB HB	0	70	30	Half Circle	Round Double, Heavy body
PPL202040DN	POLYPROPYLENE BU 2-0 X 90-40MM HC RB DN	2-0	90	40	Half Circle	Round Body, Double Needle
PPL20841	POLYPROPYLENE BU 2-0 X 70-30MM HC RB	2-0	70	30	Half Circle	Round Body
PPL508802	POLYPROPYLENE BU 5-0 X 75-11MM HC RB DN	5-0	75	11	Half Circle	Round Body, Double Needle
PPL608207	POLYPROPYLENE BU 6-0 X 60-10MM HC RB DN	6-0	60	10	Half Circle	Round Body, Double Needle
PPL608305BL	POLYPROPYLENE BU 6-0 X 60-08MM CU RB DN V	6-0	60	8	3/8 Circle	Round Body, Double Needle, Visible Black
PPL608906	POLYPROPYLENE BU 6-0 X 75-10MM HC RB DN	6-0	75	10	Half Circle	Round Body, Double Needle,
PPL702070	POLYPROPYLENE BU 7-0 X 60-10MM CU RB DN	7-0	60	10	3/8 Circle	Round Body, Double Needle,
PPL702076	POLYPROPYLENE BU 7-0 X 60-6MM CU RB DN	7-0	60	6	3/8 Circle	Round Body, Double Needle,
PPL702078BX	POLYPROPYLENE BU 7-0 X 75-8MM CU RB DN VB	7-0	75	8	3/8 Circle	Round Body, Double Needle, Visible Black,



## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL702079	POLYPROPYLENE BU 7-0 X 60-9MM CU RB DN	7-0	60	9	3/8 Circle	Round Body, Double Needle
PPL702079XL	POLYPROPYLENE BU 7-0 X 75-9MM CU RB DN	7-0	75	9	3/8 Circle	Round Body, Double Needle,
PPL708121BL	POLYPROPYLENE BU 7-0 X 60-9MM CU TC DN VB	7-0	60	9	3/8 Circle	Taper Cut, Double Needle, Visible Black
PPL408831	POLYPROPYLENE BU 4-0 X 90-26MM HC RB	4-0	90	26	Half Circle	Round Body,
PPL508710BL	POLYPROPYLENE BU 5-0 X 75-13MM HC RB DNV	5-0	75	13	Half Circle	Round Body, Double Needle, V
PPL708207	POLYPROPYLENE BU 7-0 X 60-6.5MM CU RB DN	7-0	60	6.5	3/8 Circle	Round Body, Double Needle
PPL805662	POLYPROPYLENE BU 8-0 X 60-6.5MM CU RB DN	8-0	60	6.5	3/8 Circle	Round Body, Double Needle
PPL200988	POLYPROPYLENE BU 2-0 X 90-90MM CU CT	2-0	90	90	3/8 Circle	Cutting Needle
PPL308558DN	POLYPROPYLENE BU 3-0 X 90-17MM HC RB DN	3-0	90	17	Half Circle	Round Body, Double Needle
PPL20844Z	POLYPROPYLENE BU 2-0 X 90-30MM HC RB-3D	2-0	90	30	Half Circle	Round Body
PPL40870Z	POLYPROPYLENE BU 4-0 X 70-16MM CU CT-3D	4-0	70	16	3/8 Circle	Cutting Needle
PPL50881Z	POLYPROPYLENE BU 5-0 X 70-16MM CU RB-3D	5-0	70	16	3/8 Circle	Round Body
PPL308549Z	POLYPROPYLENE BU 3-0X 90-16MM CU RC-3D	3-0	90	16	3/8 Circle	Reverse Cutting
PPL508556Z	POLYPROPYLENE BU 5- 0X90-16MM HCRBDN-3D	5	90	16	Half Circle	Round Body, Double Needle,
PPL408557Z	POLYPROPYLENE BU 4-0 X 90-16MM HCRBDN-3D	4-0	90	16	Half Circle	Round Body, Double Needle
PPL408683Z	POLYPROPYLENE BU 4-0 X 45-20MM CU RC-3D	4-0	45	20	3/8 Circle	Reverse Citting
PPL208977Z	POLYPROPYLENE BU 2-0 X 90-26MM HC TC DN-3D	2-0	90	26	Half Circle	Taper Cut, Double Needle
PPL408831Z	POLYPROPYLENE BU 4-0 X90-26MM HCRB -3D	4-0	90	26	Half Circle	Round Body,
PPL20295Z	POLYPROPYLENE BU 2-0 X90-31MM HC RB VB-3D	2-0	90	31	Half Circle	Round Body, Visible Black
PPL408761DN	POLYPROPYLENE BU 4-0 X 90-20MM HC RB DN	4-0	90	20	Half Circle	Round Body, Double Needle
PPL707237	POLYPROPYLENE BU 7-0 X 75-9.3MM CU TC DN	7-0	75	9.3	3/8 Circle	Taper Cut, Double Needle
PPL208926	POLYPROPYLENE BU 2-0 X 90-26MM HC TC	2-0	90	26	Half Circle	Taper Cut
PPL408762	POLYPROPYLENE BU 4-0 X 90-22MM HC RB	4-0	90	22	Half Circle	Round Body
PPL608726SL	POLYPROPYLENE BU 6-0 X 45-13MM CURB DN	6-0	45	13	3/8 Circle	Round Body, Double Needle
PPL608907	POLYPROPYLENE BU 6-0 X 75-9MM CU RB DN	6-0	75	9	3/8 Circle	Round Body, Double Needle
PPL408204SL	POLYPROPYLENE BU 4-0 X 60-13MM HC RB DN	4-0	60	13	Half Circle	Round Body, Double Needle
PPL708704RB	POLYPROPYLENE BU 7-0 X 60-9.5MM CU RB DN	7-0	60	9.5	3/8 Circle	Round Body, Double Needle
PPL208931	POLYPROPYLENE BU 2-0 X 90-30MM HC RC	2-0	90	30	Half Circle	Reverse Cutting
PPL408558DN	POLYPROPYLENE BU 4-0 X 90-17MM HC RB DN	4-0	90	17	Half Circle	Round Body, Double Needle
PPL208400	POLYPROPYLENE BU 2-0 X 75-70MM ST RB DN	2-0	75	70	Straight Needle	Round Body, Double Needle
PPL408665	POLYPROPYLENE BU 4-0 X 75-13MM HC TC DN	4-0	75	13	Half Circle	Taper Cut, Double Needle



## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL408329	POLYPROPYLENE BU 4-0 X 90-17MM HC RBDNVB	4-0	90	17	Half Circle	Round Body, Double Needle, Visible Black
PPL707237BL	POLYPROPYLENE BU 7-0 X 75-9MM CU TC DN VB	7-0	75	9	3/8 Circle	Taper Cut, Double Needle, Visible Black
PPL108037	POLYPROPYLENE BU 0 X 75-37MM HC TC	0	75	37	Half Circle	Taper Cut
PPL108038	POLYPROPYLENE BU 0 X 75-38MM CU CT	0	75	38	3/8 Circle	Cutting Needle
PPL20844DN	POLYPROPYLENE BU 2-0 X 90-30MM HC RB DN	2-0	90	30	Half Circle	Round Body, Double Needle
PPL405742	POLYPROPYLENE BU 4-0 X 45-13MM CU RC	4-0	45	13	3/8 Circle	Reverse Cutting
PPL308021	POLYPROPYLENE BU 3-0 X 45-26MM CU CT	3-0	45	26	3/8 Circle	Cutting Needle,
PPL702076XL	POLYPROPYLENE BU 7-0 X 75-6MM CU RB DN	7-0	75	6	3/8 Circle	Round Body, Double Needle
PPL608305XL	POLYPROPYLENE BU 6-0 X 75-9MM CU RB DN VB	6-0	75	9	3/8 Circle	Round Body, Double Needle, Visible Black
PPL608807XL	POLYPROPYLENE BU 6-0 X 75-9MM CU TC DN	6-0	75	9	3/8 Circle	Taper Cut, Double Needle
PPL608597BXL	POLYPROPYLENE BU 6-0 X 75-11MM CU RB DB VB	6-0	75	11	3/8 Circle	Round Body, D, Visible Black
PPL608814BL	POLYPROPYLENE BU 6-0 X 75-13MM CU TC DB VB	6-0	75	13	3/8 Circle	Taper Cut, D, Visible Black
PPL808732BXL	POLYPROPYLENE BU 8-0 X 75-6.5MM CURB DN VB	8-0	75	6.5	3/8 Circle	Round Body, Double Needle, Visible Black
PPL608802BXL	POLYPROPYLENE BU 6-0 X 75-11MM CUTC DN VB	6-0	75	11	3/8 Circle	Taper Cut, Double Needle, Visible Black
PPL508720BL	POLYPROPYLENE BU 5-0 X 90-13MM CUTC DN VB	5-0	90	13	3/8 Circle	Taper Cut, Double Needle, Visible Black
PPL300626	POLYPROPYLENE BU 3-0 X 75-36MM CU CT	3-0	75	36	3/8 Circle	Cutting Needle
PPL208690	POLYPROPYLENE BU 2-0 X 100-40MM CU CT	2-0	100	40	3/8 Circle	Cutting Needle
PPL108470XL	POLYPROPYLENE BU 0 X 100-40MM HC RC	0	100	40	Half Circle	Reverse Cutting
PPL018576	POLYPROPYLENE BU 1 X 100-48MM HC RC	1	100	48	Half Circle	Reverse Cutting
PPL607816DN	POLYPROPYLENE BU 6-0 X 75-8MM HC RB DN	6-0	75	48mm	Half Circle	Round Body, Double Needle
PPL707817DN	POLYPROPYLENE BU 7-0 X 60-8MM HC RB DN	7-0	60	48mm	Half Circle	Round Body, Double Needle
PPL50 8802	POLYPROPYLENE BU 5-0 X 75-11MM HC RBDN	5-0	75	11mm	Half Circle	Round Body, Double Needle
PPL100788DN	POLYPROPYLENE BU 10-0 X 20-13MM QC TC DN	10-0	20	13mm	1/4 Needle Shape	Taper Cut, Double Needle
PPL408683	POLYPROPYLENE BU 4-0 X 45-20MM CU RC	4-0	45	19mm	3/8 Circle	Reverse Cutting
PPL308663	POLYPROPYLENE BU 3-0 X 45-24MM CU RC	3-0	45	24mm	3/8 Circle	Reverse Cutting
PPL208026	POLYPROPYLENE BU 2-0 X 45-26MM CU RC	2-0	45	26mm	3/8 Circle	Reverse Cutting
PPL40 8204SL	POLYPROPYLENE BU 4-0 X 60-13MM HC RB DN	4-0	60	13mm	Half Circle	Round Body, Double Needle
PPL508557DN	POLYPROPYLENE BU 5-0 X 75-17MM HC RC DN	5-0	75	17mm	Half Circle	Reverse Cutting, Double Needle
PPL408575	POLYPROPYLENE BU 4-0 X 75-17MM HC RC	4-0	75	17mm	Half Circle	Reverse Cutting

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO


### FILAPROP SURGICAL SUTURE

PPL208833	POLYPROPYLENE BU 2-0 X 75-26MM HC RB	2-0	75	26mm	Half Circle	Round Body
PPL108026	POLYPROPYLENE BU 0 X 75-26MM HC RB	0 X	75	26mm	Half Circle	Round Body
PPL208423	POLYPROPYLENE BU 2-0 X 75-36MM HC RB	2-0	75	36mm	Half Circle	Round Body
PPL108424	POLYPROPYLENE BU 0 X 75-36MM HC RB	0 X	75	36mm	Half Circle	Round Body
PPL108434	POLYPROPYLENE BU 0 X 75-40MM HC RB	0 X	75	40mm	Half Circle	Round Body
PPL308512	POLYPROPYLENE BU 3-0 X 75-31MM CU RC	3-0	75	31mm	3/8 Circle	Reverse Cutting
PPL207697	POLYPROPYLENE BU 2-0 X 75-31MM CU RC	2-0	75	31mm	3/8 Circle	Reverse Cutting
PPL40 8521	POLYPROPYLENE BU 4-0 X 90-26MM HC RB DN	4-0	90	26mm	Half Circle	Round Body, Double Needle
PPL708705	POLYPROPYLENE BU 7-0 X 60-9.3MM CU TC DN	7-0	60	9,3mm	3/8 Circle	Taper Cut, Double Needle
PPL608814	POLYPROPYLENE BU 6-0 X 75-13MM CU TC DN	6-0	75	13mm	3/8 Circle	Taper Cut, Double Needle
PPL30 8522	POLYPROPYLENE BU 3-0 X 90-26MM HC RB DN	3-0	75	26mm	Half Circle	Round Body, Double Needle
PPL208833	POLYPROPYLENE BU 2-0 X 75-26MM HC RB	2-0	75	26mm	Half Circle	Round Body
PPL70 8735	POLYPROPYLENE BU 7-0 X 60-8MM CU RB DN	7-0	60	8mm	3/8 Circle	Round Body, Double Needle
PPL808740DN	POLYPROPYLENE BU 8-0 X 45-8MM CU RC	8-0	45	8mm	3/8 Circle	Reverse Cutting
PPL80 8088	POLYPROPYLENE BU 8-0 X 60-8MM CU RB DN	8-0	60	8mm	3/8 Circle	Round Body, Double Needle
PPL70 8702	POLYPROPYLENE BU 7-0 X 60-9.3MM CU RB DN	7-0	60	9,3mm	3/8 Circle	Round Body, Double Needle
PPL70 2079XL	POLYPROPYLENE BU 7-0 X 75-9.3MM CU RB DN	7-0	75	9,3mm	3/8 Circle	Round Body, Double Needle
PPL608807	POLYPROPYLENE BU 6-0 X 60-9.3MM CU TC DN	6-0	60	9,3mm	3/8 Circle	Taper Cut, Double Needle
PPL308525DN	POLYPROPYLENE BU 3-0 X 90-31MM HC RB DN	3-0	90	31mm	Half Circle	Round Body, Double Needle
PPL408355	POLYPROPYLENE BU 4-0 X 90-26MM HC RB	4-0	75	26mm	Half Circle	Round Body
PPL508934	POLYPROPYLENE BU 5-0 X 90-17MM HC TC DN	5-0	90	17mm	Half Circle	Taper Cut, Double Needle
PPL40 8935	POLYPROPYLENE BU 4-0 X 90-17.5MM HC TC	4-0	90	17mm	Half Circle	Taper Cut
PPL40 8845	POLYPROPYLENE BU 4-0 X 90-26MM HC TC DN	4-0	90	26mm	Half Circle	Taper Cut, Double Needle
PPL308976	POLYPROPYLENE BU 3-0 X 90-26MM HC TC DN	3-0	90	26mm	Half Circle	Taper Cut, Double Needle
PPL20 8977	POLYPROPYLENE BU 2-0 X 90-26MM HC TC DN	2-0	90	26mm	Half Circle	Taper Cut, Double Needle
PPL40 8526DN	POLYPROPYLENE BU 4-0 X 75-22MM HC RB DN	4-0	75	22mm	Half Circle	Round Body, Double Needle
PPL308771	POLYPROPYLENE BU 3-0 X 75-22MM HC RB DN	3-0	75	22mm	Half Circle	Round Body, Double Needle
PPL209523	POLYPROPYLENE BU 2-0 X 75-22MM HC RB DN	2-0	75	22mm	Half Circle	Round Body, Double Needle
PPL308558	POLYPROPYLENE BU 3-0 X 91-17MM HC RB	3-0	75	17mm	Half Circle	Round Body
PPL408558SL	POLYPROPYLENE BU 4-0 X 75-17MM HC RB	4-0	75	17mm	Half Circle	Round Body
PPL308558DN	POLYPROPYLENE BU 3-0 X 75-17MM HC RB DN	3-0	75	17mm	Half Circle	Round Body, Double Needle
PPL509702	POLYPROPYLENE BU 5-0 X 60-9.3MM CU RC	5-0	60	9,3mm	3/8 Circle	Reverse Cutting

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

PPL405742XL	POLYPROPYLENE BU 4-0 X 76-13MM CU RC	4-0	75	13mm	3/8 Circle	Reverse Cutting
PPL408558SL	POLYPROPYLENE BU 4-0 X 75-17MM HC RB	4-0	75	17mm	Half Circle	Round Body
PPL708705	POLYPROPYLENE BU 7-0 X 60-9MM CU TC DN	7-0	60	9,3mm	3/8 Circle	Taper Cut, Double Needle
PPL608306DN	POLYPROPYLENE BU 6-0 X 75-13MM CU RC DN	6-0	75	9,3mm	3/8 Circle	Reverse Cutting, Double Needle
PPL70 8702	POLYPROPYLENE BU 7-0 X 60-9.3MM CU RB DN	7-0	60	9,3mm	3/8 Circle	Round Body, Double Needle
PPL708703DN	POLYPROPYLENE BU 7-0 X 76-9.3MM CU RC	7-0	75	9,3mm	3/8 Circle	Reverse Cutting
PPL60 8726XL	POLYPROPYLENE BU 6-0 X 75-13MM CU RB DN	6-0	75	13mm	3/8 Circle	Round Body, Double Needle
PPL607406	POLYPROPYLENE BU 6-0 X 75-9.3MM CU RB DN	6-0	75	9,3mm	3/8 Circle	Round Body, Double Needle
PPL708727DN	POLYPROPYLENE BU 7-0 X 60-13MM CU RC DN	7-0	60	13mm	3/8 Circle	Reverse Cutting, Double Needle
PPL708733	POLYPROPYLENE BU 7-0 X 75-8MM CU RB DN	7-0	75	13mm	3/8 Circle	Round Body, Double Needle
PPL708727DN	POLYPROPYLENE BU 7-0 X 60-13MM CU RC DN	7-0	60	13mm	3/8 Circle	Reverse Cutting, Double Needle
PPL608712	POLYPROPYLENE BU 6-0 X 60-9.3MM CU RB DN	6-0	60	9,3mm	3/8 Circle	Round Body, Double Needle
PPL608697	POLYPROPYLENE BU 6-0 X 45-11MM CU SM	6-0	45	11mm	3/8 Circle	Spatulaed Musles Needle

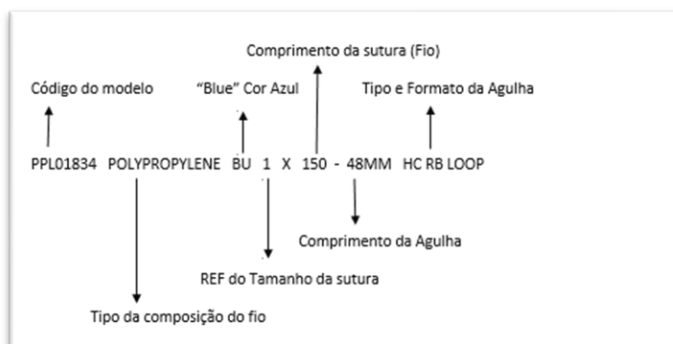

## II - INDICAÇÃO, FINALIDADE OU USO A QUE SE DESTINA O PRODUTO MÉDICO

### INDICAÇÕES DE USO

Os fios de sutura FILAPROP™ destinam-se à coaptação e/ou laqueação dos tecidos moles em geral, inclusive nas intervenções cardiovasculares, oftalmológicas e neurológicas.

Devido ao fato de o material ser praticamente inerte do ponto de vista biológico, recomenda-se a sua utilização nos casos que impliquem uma reação mínima dos tecidos aos fios de sutura. Devido à baixa aderência aos tecidos, o fio de sutura FILAPROP™ é indicado nos casos que requerem a ablação posterior dos fios de sutura.

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

#### APLICAÇÃO

A seleção da sutura para implantação depende do estado do doente, da experiência do cirurgião, da técnica cirúrgica, assim como do tamanho da lesão.

#### ATUAÇÃO

Os fios de sutura FILASILK conduzem a uma reação inflamatória aguda nos tecidos, seguida de um envolvimento gradual do fio por tecido conjuntivo fibroso. Embora os fios de sutura de seda não sejam absorvíveis, a degradação progressiva da fibra proteínica da seda in vivo pode resultar na diminuição gradual da força tênsil dos fios de sutura ao longo do tempo.

#### CONTRA-INDICAÇÕES

Nenhuma conhecida

#### ADVERTÊNCIAS/PRECAUÇÕES/INTERAÇÕES

##### AVISOS

- Os cirurgiões devem considerar o desempenho in vivo (na seção 'DESEMPENHO') e devem estar familiarizados com os procedimentos cirúrgicos e técnicas que envolvem suturas não absorvíveis antes de utilizar a sutura FILAPROP™ para fechamento da ferida, pelo o risco de deiscência da ferida poder variar com o local de aplicação e os materiais de sutura utilizados.
- Na cirurgia das vias urinárias ou biliares, deve ser tido cuidado para evitar o contato prolongado da sutura com soluções de sal, para evitar a formação de cálculos.

#### PRECAUÇÕES


- Deve ser seguida a prática cirúrgica aceitável para o tratamento de feridas contaminadas ou infectadas.
- No tratamento deste material de sutura, deve-se tomar cuidado para evitar danos. Evite amassar ou dobrar devido à utilização de instrumentos cirúrgicos, tais como pinças ou suportes de agulhas. A segurança adequada dos nós requer a técnica

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

cirúrgica aceitável para nós planos e quadrados com laçada adicional conforme indicado pelas circunstâncias cirúrgicas e pela experiência do cirurgião.

- Devem ser tomados cuidados para evitar danos durante o manuseio de agulhas cirúrgicas. Agarrar a agulha numa área entre um terço ( $1/3$ ) e a metade ( $1/2$ ) da distância a partir da tremidade de fixação da tremidade aguçada. Agarrar na tremidade aguçada poderia prejudicar o desempenho de penetração e causar quebra da agulha. Agarrar na zona posterior poderia levar a agulha a soltar-se ou a sofrer flão ou quebra. A tentativa de endireitar as agulhas pode enfraquece-las e torna-las a ser menos resistente à flão e quebra.

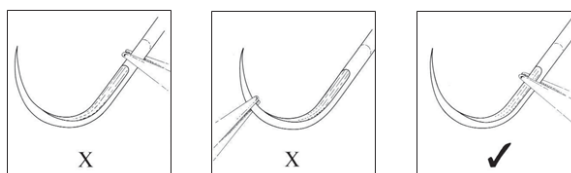

- Os utilizadores devem ter cuidado ao manusear agulhas cirúrgicas para evitar lesões por picada inadvertida. Descartar as agulhas usadas de forma adequada.

## REAÇÕES ADVERSAS

### INFORMAÇÕES ÚTEIS PARA EVITAR DETERMINADOS RISCOS DECORRENTES DA UTILIZAÇÃO DO PRODUTO MÉDICO

- As reações adversas associadas à utilização de FILAPROP™ incluem irritação local transitória no local da ferida ou resposta inflamatória transitória ao corpo estranho. Como todos os corpos estranhos, FILAPROP™ pode contribuir para o aumento de uma infecção istente.

## ESTERILIZAÇÃO

- As suturas FILAPROP™ são esterilizadas por óxido de etileno (EtO), como indicado na embalagem. Não reesterilizar. Não utilizar se a embalagem estiver aberta ou danificada. Descartar as restantes suturas abertas.

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

#### ARMAZENAMENTO

- Condições de armazenamento recomendadas: temperatura inferior a 25 °C, afastado de umidade e do calor direto. Não utilizar depois de pirada a validade.

### III – FORMA DE APRESENTAÇÃO DE PRODUTO MÉDICO

#### EMBALAGEM

O fio de sutura FILAPROP está disponível no seguinte comprimento 13, 45, 60, 70, 75, 90, 100, 150. O fio de sutura FILAPROP é estéril, se a embalagem não estiver danificada nem tiver sido aberta.

A embalagem inclui:

12, 24 ou 36 Suturas Cirúrgicas (quantidade conforme tabela de modelos)

01 Instruções de Uso

#### MATERIAL DE COMPOSIÇÃO

**Tabela 01:** Materiais de Fabricação

Nome da peça	Material
Fio	Polipropileno
Corante do Fio (somente modelo violeta)	Azul Ftalocianina (Número do Índice de Cor: 74160)
Agulha	Aço Inoxidável _série 300_ASI 302



## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

FILAPROP SURGICAL SUTURE

### SÍMBOLOS USADOS NA ROTULAGEM



### INFORMAÇÕES DE PRODUTO DEVOLVIDO

Para informações detalhadas sobre a Política Meril de Devolução do Produto, entrar em contato com o representante local.

### DIMENSÕES

Ver tabela de modelos

## INSTRUÇÕES DE USO DE PRODUTO MÉDICO

### FILAPROP SURGICAL SUTURE

**ATENÇÃO:** Conteúdo Estéril e Apirogênico se a embalagem não estiver aberta ou danificada.

Esterilizado por: Óxido de Etileno

Validade: 5 anos após a esterilização

### PRODUTO MÉDICO HOSPITALAR

### PROIBIDO REPROCESSAR



#### Fabricante Internacional

Meril Endo Surgery Pvt. Ltda  
Survey nº 135/2/B e 174/2, 3º andar, E1-E3,  
Meril Park \_ Bilakhia Muktanand Marg, Chala, Vapi,  
Dist: Valsad – 396191 Gujarat, India  
[www.merillife.com](http://www.merillife.com)

#### Importador no Brasil

DOC MED Comércio Importação E Exportação Ltda.  
CNPJ: 66.877.184/0001-80  
End: Alameda dos Maracatins nº 992, 8º andar, São  
Paulo, SP  
Tel: (11) 2506-3624  
[www.merillife.com](http://www.merillife.com)

#### Responsável legal

Sr. RAKESH KUMAR KANOJIA  
(Sócio Representante Legal)

#### Responsável Técnico

DR. CAIO DE MACEDO MATOS  
CRF/SP: 49805